beurer BY 20   

Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany
www.beurer.com • www.beurer-healthguide.com
www.beurer-gesundheitsratgeber.com

DEUTSCH

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ENGLISH

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution	IP 22	Protected against solid foreign objects 12.5 mm in diameter and larger, and against drops of water when the housing is angled up to 15°
	Follow instructions for use	LOT	Batch designation
	Type BF applied part		Permissible storage and transport temperature and humidity
	Disposal in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment)		Permissible operating temperature and humidity
	Manufacturer	CE ₀₄₈₃	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical devices.

Included in delivery: • Thermometer • Instruction for Use • 1 x 1,5V LR 41 battery

Important safety information – retain for later use.

In order to make optimum use of all the merits of the clinical thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.

- The thermometer is intended to measure the temperature of the human body.
- Do not allow children to play with the device. Children must be supervised at all times whilst using the device.
- The comfortor thermometer is intended only for oral measurement of body temperature in children up to 5 years of age. It is not suitable for constant sucking.
- Before every use, check the comfortor thermometer and especially the silicon nipple for sins of damage or wear. Damaged or worn comfortor thermometers should not be used.
- Check that the teats are firmly attached to the pacifier. There is a choking hazard.
- The clinical thermometer contains an alkaline button cell (LR 41) suf- ficient for approx. 1200 measurements.
- Under specific conditions, the displayed temperature may be considerably different from the actual body temperature of the child (-0.5°C to 0.5°C). Therefore, please ensure that the child does not eat or drink anything shortly before the measurement, and that the child is not crying or breathing through the mouth when the temperature is being measured.
- In case of doubt and signs of fever (temperature higher than 38.5°C, flushed appearance, heat, perspiration, restlessness) measure the body temperature afterwards by using a conventional clinical thermometer in the anus (rectal) and discuss the measured temperatures with the physician.

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- Always wait for the acoustic signal which indicates the minimum time required for measurement. Discuss the temperature with your family physician.
- The thermometer contains sensitive electronic parts. Therefore, protect it from blows, bending, high temperature or direct exposure to sunlight.
- Keep the thermometer away from solvents and other chemicals.
- When the thermometer is switched on, it carries out a self-test. Therefore the accuracy of measurement need not be tested at regular intervals.
- This unit is in line with European Standard EN 60601-1-2 (In accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-8) and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. For more details, please contact customer service at the address indicated.

The pacifier is delivered in one size. Before using it for the first time, check that the pacifier fits into the mouth of your child. If the pacifier is too large, there is a choking hazard. Press the ON switch to turn the unit on. A short acoustic signal will announce that it is switched on. The thermometer will Fig. 1 carry out a self-test for about 2 seconds. During the self-test all segments of the display will be visible (Fig. 1). Then „L0“°C or the value of the last measurement will appear in the display. Make sure the baby keeps the comforter in the mouth during the measurement and breaths calmly through the nose. Then the current temperature is immediately displayed. The measurement takes approximately 5 minutes, which is indicated by an acoustic signal. By prolonging the length of measurement after the acoustic signal, more accurate measurements can be obtained. If temperatures higher than 42.9°C are measured, the symbol for too high temperature, „Hr“ will appear. „L0“ appears for temperatures lower than 32°C. In order to increase the life of the battery, please switch off the thermometer by briefly pressing the ON key after the measurement.

Otherwise the thermometer will automatically switch off after about 10 minutes.

Memory

The last measured value will be stored automatically. To display the stored measurement value, press the ON button. Next to the measured value the memory symbol (M) will appear.

Changing the battery/Disposal

The clinical thermometer contains an alkaline button cell (LR 41) suf- ficient for approx. 1200 measurements.

For safety reasons, the battery cannot be exchanged. When the battery symbol (■) appears on the display, please dispose the clinical thermometer via your local dealer for electric goods or your local depot for special waste. Do not dismantle the device.

The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany
www.beurer.com • www.beurer-healthguide.com
www.beurer-gesundheitsratgeber.com

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